## EmbryoGlue®

### US

#### EN: Indication for use

Medium for embryo transfer.

#### Product Description

EmbryoGlue® is a bicarbonate buffered medium containing recombinant human albumin, hyaluronan and gentamicin as an antibacterial agent.

For use after equilibration at +37°C and 6 % CO2 atmosphere.

#### Storage instructions and stability

Store dark at +2 to +8 °C.

EmbryoGlue® is stable until the expiry date shown on the container labels and the LOT-specific Certificate of Analysis.

**Media bottles should not be stored after opening. Discard excess media after completion of the procedure.**

#### Directions for use

##### Preparation of dishes

Add no less than 0.5 ml of EmbryoGlue® to the well and no less than 0.5 ml to the most of a 1-well dish. Equilibrate the dish at +37°C and 6 % CO2 for 2–24 hours with the lid on.

In a non-humidified environment, dishes should not be stored for > 6 h.

##### Preparation of embryos

Equilibrate the embryos in EmbryoGlue® for a minimum of 10 minutes at +37 °C in a 6% CO2 environment prior to transfer. The embryos can stay in EmbryoGlue® for up to 4 hours. To avoid evaporation, keep the lid on the dish.

In a non-humidified environment, complete the embryo transfer procedure within 6 h after dish preparation.

##### Preparation of transfer catheter

Attach the transfer catheter to a 1 ml, non-toxic syringe and draw up approximately 0.5 ml of the equilibrated medium from the moat. Rinse the catheter by expelling EmbryoGlue® back into the moat.

##### Embryo transfer

After rinsing, draw approximately 0.1 mL of EmbryoGlue® from the center well and expel into the moat until approximately 20 µL is left in the syringe. Under microscopic control, gently load the embryos into the catheter in approximately 5–10 µL of additional EmbryoGlue® followed by a small amount of air. (The small pocket of air at the tip allows better imaging of the tip for ultrasound guided transfers). For the embryo transfer, pass the tip of the catheter into the uterus approximately 1 cm from the top of the cavity and expel the embryos in a total volume of approximately 25–30 µL of medium. **Slowly withdraw the catheter while maintaining steady pressure on the plunger of the syringe.** Make a final microscopic examination of the catheter.

<b>Specifications</b>	
Sterile filtered	SAL 10 <sup>+</sup>
Mouse embryo assay (1-cell) [% expanded blastocyst within 96 hours]	≥ 80
Bacterial endotoxins (LAL assay) [EU/mL]	< 0.25
LOT specific test results are available on the Certificate of Analysis provided with each delivery.	

<b>Precautions</b>	
Discard product if bottle integrity is compromised. Do not use EmbryoGlue® if it appears cloudy.	

EmbryoGlue® contains recombinant human albumin (produced in yeast), hyaluronan and gentamicin. Do not use in patients with known hypersensitivity/allergy to any of the components.

Re-use may result in microbiological contamination and/or product changes in the product.

To avoid contamination Vitrolife strongly recommends that media should be opened and used only with aseptic technique.

The risk of reproductive toxicity and developmental toxicity for IVF media, including Vitrolife’s IVF media, have not been determined and are uncertain.

Not for injection.

<b>Description of ISO Symbols</b>	
<span><span><span></span></span></span> Sterilized using aseptic processing techniques	
<span><span><span></span></span></span> Temperature limit	
<span><span><span></span></span></span> Date of manufacture	
<span><span><span></span></span></span> Do not re-use, discard after procedure.	
<span><span><span></span></span></span> Use by - see label.	
<span><span><span></span></span></span> Caution: Consult accompanying documents.	
<span><span><span></span></span></span> Catalog number	
<span><span><span></span></span></span> Batch code	
<span><span><span></span></span></span> CE Mark (Conformité Européen)	

## CHINA

<b>EN: Indication for use</b>	
Medium for embryo transfer.	

<b>Product Description</b>	
EmbryoGlue® is a bicarbonate buffered medium containing recombinant human albumin, hyaluronan and gentamicin as an antibacterial agent.	
For use after equilibration at +37°C and 6 <span> </span> % CO2 atmosphere.	

#### Storage instructions and stability

Store dark at +2 to +8 °C.

EmbryoGlue® is stable until the expiry date shown on the container labels and the LOT-specific Certificate of Analysis.

**Media bottles should not be stored after opening. Discard excess media after completion of the procedure.**

#### Directions for use

##### Preparation of dishes

Add no less than 0.5 ml of EmbryoGlue® to the well and no less than 0.5 ml to the most of a 1-well dish. Equilibrate the dish at +37°C and 6 % CO2 for 2–24 hours with the lid on.

In a non-humidified environment, dishes should not be stored for > 6 h.

##### Preparation of embryos

Equilibrate the embryos in EmbryoGlue® for a minimum of 10 minutes at +37 °C in a 6% CO2 environment prior to transfer. The embryos can stay in EmbryoGlue® for up to 4 hours. To avoid evaporation, keep the lid on the dish.

In a non-humidified environment, complete the embryo transfer procedure within 6 h after dish preparation.

##### Preparation of transfer catheter

Attach the transfer catheter to a 1 mL, non-toxic syringe and draw up approximately 0.5 mL of the equilibrated medium from the moat. Rinse the catheter by expelling EmbryoGlue® back into the moat.

##### Embryo transfer

After rinsing, draw approximately 0.1 mL of EmbryoGlue® from the center well and expel into the moat until approximately 20 µL is left in the syringe. Under microscopic control, gently load the embryos into the catheter in approximately 5–10 µL of additional EmbryoGlue® followed by a small amount of air. (The small pocket of air at the tip allows better imaging of the tip for ultrasound guided transfers). For the embryo transfer, pass the tip of the catheter into the uterus approximately 1 cm from the top of the cavity and expel the embryos in a total volume of approximately 25–30 µL of medium. **Slowly withdraw the catheter while maintaining steady pressure on the plunger of the syringe.** Make a final microscopic examination of the catheter.

<b>Specifications</b>	
Sterile filtered	SAL 10 <sup>+</sup>
Mouse embryo assay (1-cell) [% expanded blastocyst within 96 hours]	≥ 80
Bacterial endotoxins (LAL assay) [EU/mL]	< 0.25
LOT specific test results are available on the Certificate of Analysis provided with each delivery.	

<b>Precautions</b>	
Discard product if bottle integrity is compromised. Do not use EmbryoGlue® if it appears cloudy.	

EmbryoGlue® contains recombinant human albumin (produced in yeast), hyaluronan and gentamicin. Do not use in patients with known hypersensitivity/allergy to any of the components.

Re-use may result in microbiological contamination and/or product changes in the product.

To avoid contamination Vitrolife strongly recommends that media should be opened and used only with aseptic technique.

The risk of reproductive toxicity and developmental toxicity for IVF media, including Vitrolife’s IVF media, have not been determined and are uncertain.

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<span><span><span></span></span></span> Sterilized using aseptic processing techniques	
<span><span><span></span></span></span> Temperature limitation	
<span><span><span></span></span></span> Date of manufacture	
<span><span><span></span></span></span> Do not re-use, discard after procedure.	
<span><span><span></span></span></span> Use by - see label.	
<span><span><span></span></span></span> Caution: Consult accompanying documents.	
<span><span><span></span></span></span> Catalog number	
<span><span><span></span></span></span> Batch code	
<span><span><span></span></span></span> CE Mark (Conformité Européenne)	

<b>EN: Indication for use</b>	
Medium for embryo transfer.	

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<b>Storage instructions and stability</b>	
Store dark at +2 to +8 <span> </span> °C.	
EmbryoGlue® is stable until the expiry date shown on the container labels and the LOT-specific Certificate of Analysis.	
<b>Media bottles should not be stored after opening. Discard excess media after completion of the procedure.</b>	

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