# RapidWarm™ Blast

Vitrolife

US

EN: Indication for use

RapidWarm™ Blast is indicated for warming of vitrified human blastocyst stage embryos.

**Product Description**

Med for warming of vitrified human blastocyst stage embryos.

**SUPPLEMENTED WITH HSA**

RapidWarm™ Blast contains three media for the warming of vitrified human blastocyst stage embryos.

The media consists of a MOPS buffered solution containing gentamicin as an antibiotic agent and human serum albumin.

Warm 1™ Blast contains sucrose as cryoprotectant.

Warm 2™ Blast contains sucrose as cryoprotectant.

Warm 3™ Blast contains no cryoprotectants.

For use after warming to +37 °C.

**Storage instructions and stability**

Store dark at +2 to +4 °C.

RapidWarm™ Blast is stable until the expiry date shown on the bottle labels and the LOT specific Certificate of Analysis.

Media bottles should not be stored after opening. Discard excess media after completion of procedure.

**Use of storage device**

Use a legally marketed storage device indicated for use in human embryo vitrification procedures, in order to assure sufficient cooling and warming rates.

Use a closed storage system to prevent the potential risk of viral or other contamination of samples.

Perform the actual vitrification and warming according to the Instructions for Use for the storage device.

**Directions for use**

Caution: Read the label and understand the directions for use. Warnings and Precautions, and be trained in the correct procedure before using Vitrolife RapidWarm™ Blast and RapidWarm™ Blast kits for vitrification and warming of human blastocyst stage embryos.

The following is the general procedure for using RapidWarm™ Blast.

Note that the timing has to be accurate.

**Warnings:**

Place 0.5-1 ml of each of the following media into separate wells of a 4-well plate and warm to 37 °C:

• Warm 1™ Blast

• Warm 2™ Blast

• Warm 3™ Blast

All manipulations of the blastocysts are carried out at 37 °C (on the heated stage).

Remove the embryo-containing the vitrified blastocyst from the cryotrap container. Follow the warming instructions for the specific cryodevice used. Immediately after warming, transfer blastocysts into Warm 1™ Blast.

Allow the blastocysts to fall from the device and sink to the bottom.

Transfer the blastocysts into Warm 2™ Blast and let the blastocysts remain in the solution for 3 min.

Transfer the blastocysts into Warm 3™ Blast and let the blastocysts remain in the solution for 5-10 min.

Rinse the blastocysts in culture media several times and continue culture according to laboratory practice.

VitroLife recommends using G-2™ or EmbryoGleue® for this procedure.

**Specifications**

Sterile filtered SAL 10\*

Mouse Embryo Assay (1-cell) % re-expansion blastocyst 24 hours post-test

Bacterial endotoxin (LAL assay) EU/ml < 0.5

LOT-specific test results are available on the Certificate of Analysis provided with each delivery.

**Precautions**

Caution: Federal (US) law restricts this device to sale by or on the order of a physician or practitioner trained in its use.

The long-term safety of vitrification and/or blastocyst collapse on children born following this method of embryo vitrification procedure has not been established.

The risk of reproductive toxicity and developmental toxicity for IVF media, including Vitrolife's IVF media, have not been determined.

This safety and effectiveness of vitrification has not been fully evaluated in human embryos that have not yet reached the blastocyst stage of development.

Discard product if bottle integrity is compromised. Do not use if RapidWarm™ Blast is not cloudy.

RapidWarm™ Blast contains gentamicin and human serum albumin. Do not use in patients with known hypersensitivity/allergy to any of the components.

Caution: All blood products should be treated as potentially infectious. Source material from which this product was derived was found negative when tested for antibodies to HIV, HBV, HCV and HIV-1 and non-reactive for IgG antibodies to HbsAg, HCV RNA and syphilis. No known test methods can offer assurance that products derived from human blood will not transmit infectious agents.

Re-use may result in microbiological contamination and/or property changes in the product.

To avoid contamination Vitrolife strongly recommends that media should be opened and used only with aseptic technique.

No for injection.

**Description of ISO Symbols**

STERILE[A] Sterilized using aseptic processing techniques

-14°C Temperature limit

Do not reuse, discard after procedure.

Date of manufacture.

Use by - see label.

Caution: Consult accompanying documents.

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