Vacuum Pump Tubing

Vitrolife

US

en: Indication for use

For connection between the needle set and the vacuum pump.

Contraindications

None known.

Product Description

Vacuum Pump Tubing is not made with natural rubber latex, PVC and DEHP and is available with:

- Male luer lock connection and hydrophobic filter. For pump connection with OD 7-9.5 mm barb fitting.
- Male and female luer lock connections.
- Male luer lock connection and open end.

Storage Instructions

Store at +2 to +30°C.

Directions for use

The product shall be used by health care professionals within ART or gynaecology.

The patient target group is an adult or reproductive-age population that undergoes ART treatment or fertility preservation.

Check that the package is intact and the "use by" date has not expired.

Carefully open the bag containing the vacuum pump tubing.

Connect the tubing to the needle and the vacuum pump.

Specifications

Sterilized using ethylene oxide

SAL 10⁶

LOT specific test results are available on the Certificate of Analysis provided with each delivery.

Precautions

Discard if the product or the sterile packaging is damaged.

The Vacuum Pump Tubing is intended for single use only and SHALL NOT BE REUSED. Reuse may cause material degradation and/or contamination and failed procedure.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Discard product according to standard clinical practice for medical hazardous waste when the procedure is finished.

Caution: Federal (US) law restricts this device to sale by or on the order of a physician or a practitioner trained in its use (Rx only).

Symbols glossary

** Medical devices – Symbols to be used with information to be supplied by the manufacturer (ISO 15223-1:2021).

Symbol	Title	Reference and symbol ID
	Consult instructions for use	**Ref no: 5.4.3
	Caution	**Ref no: 5.4.4
	Manufacturer	**Ref no: 5.1.1
	Date of manufacture	**Ref no: 5.1.3
	Use-by date	**Ref no: 5.1.4
	Sterilised using ethylene oxide	**Ref no: 5.2.3
	Batch code	**Ref no: 5.1.5
	Catalogue number	**Ref no: 5.1.6
	Single sterile barrier system	**Ref no: 5.2.11
	Do not re-use	**Ref no: 5.4.2
	Temperature limit	**Ref no: 5.3.7
	Medical Device	**Ref no: 5.7.7
	CE Mark (Conformité Européenne)	European conformity (CE) mark with Notified Body identification number as defined in MDR 2017/745 Annex V
	Not made with natural rubber latex	**Ref no: 5.4.5 (Annex B for negation symbol)

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M1018-A/05 Vacuum Pump Tubing © Vitrolife