OVULATION PREDICTOR

INTENDED USE

OvuQuick One-Step Ovulation Test is a rapid chromatographic immunoassay for the qualitative detection of luteinizing hormone (LH) in urine to aid in the detection of ovulation. For in vitro diagnostic use only. For self testing.

PRECAUTIONS

- Read this entire pamphlet before you start the test.
- Do not use after the expiration date labeled on the side of the carton.
- Store at room temperature or refrigerated, 35–86°F (2–30°C).
- Keep out of reach of children.
- Not for internal use.
- This test cannot be used for contraception.
- Do not eat, drink or smoke in the area where the specimens or kits are handled.
- Do not use test if protective foil pouch is opened or damaged.

KIT CONTENTS

- Pouch Containing Test Cassette and Dropper (6 or 9).
- Instruction for Use.
- Plastic Cup (6 or 9).

HOW THE TEST WORKS

OvuQuick One-Step[®] is a fast, accurate and easy-to-use test to help you predict when you are likely to ovulate. It is during ovulation that you are most able to become pregnant.

WHAT IS OVULATION?

Ovulation is the release of an egg from your ovary. This makes the few days around ovulation the most likely time to become pregnant. Your body makes a substance that triggers ovulation. This substance is called luteinizing hormone (LH). You make only a little LH during most of your menstrual cycle. But, in the middle part of your cycle, the amount of LH suddenly increases for a short time. This increase is called the LH surge, and it usually causes ovulation. Ovulation usually takes place 24–36 hours after the LH surge begins. It is important to know that the LH surge and ovulation may not occur in all cycles. Menstrual bleeding that begins at the expected time is not always a sign that you have ovulated.

WHEN TO START TESTING

The OvuQuick One-Step Ovulation kit contains numerous tests, allowing you to test over several days to find your most fertile time. Calculate when to start testing using the chart.

1. First calculate the Length of your average Menstrual Cycle. Your Menstrual Cycle Length is the number of days from the first day of your period (menstrual bleeding) to the last day before your next period starts. Circle the number of days in your cycle on the "WHEN TO START TESTING CHART." Note: If your cycle is irregular, you may want to use your shortest cycle length when reading the chart.
2. Use the EXAMPLE and SAMPLE CALENDAR to determine which day you should begin testing.

WHEN TO START TESTING CHART

Cycle Length (days)	Start Test Series Beginning on	Cycle Length (days)	Start Test Series Beginning on
21	Day 4	31	Day 10
22	Day 5	32	Day 10
23	Day 6	33	Day 12
24	Day 7	34	Day 13
25	Day 8	35	Day 14
26	Day 9	36	Day 15
27	Day 9	37	Day 16
28	Day 9	38	Day 17
29	Day 9	39	Day 18
30	Day 10	40	Day 19

SAMPLE CALENDAR

S	M	T	W	T	F	S
		1	2 Period Starts Day 1	3	4	5
6	7	8	9	10 Start Testing	11	12
13	14	15	16	17	18	19
20	21	22	23	24	25	26
27	28	29	30	31		

Example:

Your usual cycle length is 28 days. Your period started on the 2nd day of the month. The "WHEN TO START TESTING CHART" shows to start testing on Day 9. **Beginning with the 2nd day**, count ahead 9 days on the calendar. You would begin testing your urine on the 10th of the month. **NOTE:** If your menstrual cycle typically exceeds 40 days, or is shorter than 21 days, you should consult your Physician as to the proper day to begin testing.

GETTING READY

Collect your urine once a day between 10 a.m. and 8 p.m. in one of the collection cups provided. For best results, collect your urine at about the same time each day. Do not collect your first urine after waking up.


Reduce your liquid intake approximately 2 hours prior to urine collection. Collect urine in the plastic cup provided. Once the urine is collected, proceed with the test.

You can store your urine for testing later in the day. Your urine can be stored at room temperature for up to 8 hours or in the refrigerator for 24 hours. Do not freeze it. For best results, test your urine on the same day it is collected. If you refrigerate your urine, let it reach room temperature before testing (about 30 minutes). Do not shake the container, allow sediment to settle. Use urine from the top of the container. You will need a watch or a clock before you begin the test.

DIRECTIONS FOR USE

Allow the test device and urine specimen to equilibrate to room temperature prior to testing.


1. Determine the day you will begin testing. (See above section "WHEN TO START TESTING").
2. Collect urine in a clean, dry collection cup (provided in kit).
3. Remove the test device and dropper from sealed pouch and use it as soon as possible.
4. Place the test device on a clean and level surface.
5. Place dropper in urine sample and draw up urine.
6. **Hold the dropper upright and transfer 3 full drops of urine to the sample well** of the test device and then start the timer. Avoid trapping air bubbles in the sample well.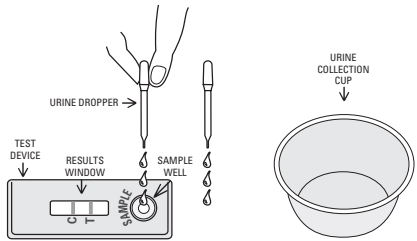


7. Start the timer and wait for the line(s) to appear. **The result should be read at 5 minutes.** Do not read the results after 10 minutes.

INTERPRETATION OF RESULTS


Positive Result: If two lines are visible and Test Line (T) is EQUAL TO OR DARKER than the Control Line (C) the result is POSITIVE. You have detected your LH surge. Ovulation should occur within 24–36 hours.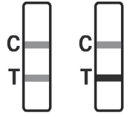

Negative Result: If two lines are visible, but the Test Line (T) is lighter than the Control Line (C), or there is no Test Line (T). This indicates that no LH surge has been detected and you should continue daily testing.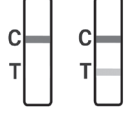

Invalid Result: Control Line (C) fails to appear. You must see a Control Line (C) after adding your urine. If you do not see a Control Line (C) the test is INVALID. Insufficient specimen volume or incorrect procedural technique are the most likely reasons for an invalid result. Review the procedure and repeat the test with a new test device. If the problem persists, discontinue using the kit immediately and contact your local distributor or Vitrolife technical support line.

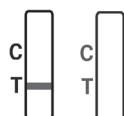HOW TO RECOGNIZE YOUR LH SURGE

- If your Test Result is **POSITIVE**, you are having an LH surge. You are most likely to ovulate sometime in the next 24–36 hours after your LH surge begins.

- No matter how many days you have an LH surge, the **first day** of your LH surge, and the next day and a half, are the days you are most likely to become pregnant.

- If your Test Result is **NEGATIVE**, you are probably not having an LH surge.

- Unless your doctor tells you differently, you can stop testing once you have detected your LH surge.

TEST LIMITATIONS

Read all the information before performing the test. Do not open the foil pouch until you are ready to start the test. Not every woman ovulates at mid-cycle, therefore, you may not see a positive result during your first 5 days of testing. Continue testing with the OvuQuick One-Step Ovulation test to detect your LH surge.

Some prescription drugs may affect the result such as menotropins for injection (Pergonal[®]) and danazol (Danocrine[®]). If you are using Clomiphene Citrate (e.g., Clomid[®] and Serophene[®]), consult your physician for possible interference with the test. Some rare medical conditions and/or the onset of menopause can cause elevated LH levels. Some women do not ovulate every cycle and will not see an increase in LH levels during these non-ovulating cycles.

PERFORMANCE CHARACTERISTICS

Laboratory studies have shown that the sensitivity of the OvuQuick LH One Step Test Device is 40 mIU/mL and the accuracy is 99%.

QUESTIONS AND ANSWERS

1. Will the amount of liquid that I drink affect the test results?
You should reduce your liquid intake for about 2 hours before you collect your urine. A diluted urine sample could prevent you from seeing your LH surge.
2. Do alcohol or common medications affect the test?
No, but you should consult your physician if you are taking any hormonal medication. Also, recent oral contraceptive use, breastfeeding or pregnancy could affect the results.
3. Can I use the test to avoid getting pregnant?
No.
4. Once I see a positive result, when is the best time to have intercourse?
Ovulation usually follows the start of the LH surge within 24–36 hours. Therefore, it is best to act on the information as soon as possible. You may want to check with your doctor for advice that is right for you.
5. I am now using the basal body temperature method (BBT). Does this test replace BBT?
You should follow your doctor's instructions about the use of BBT. The BBT method is not always accurate in predicting the time of ovulation. OvuQuick One-Step detects the hormone rise that takes place before ovulation. OvuQuick One-Step helps you to better predict when you will ovulate.
6. How important is timing when I read my test?
Timing is very important! Use your watch or clock and read your result between 5 and 10 minutes. Do not read your result after 10 minutes.
7. The control line did not appear when I ran my test; is that okay?
The Control Line tells you the test is working properly. If no control line is visible at 5 minutes, the test is invalid. Try again with a fresh sample and a new cassette; if problem persists, contact our Technical Support line.
8. Can tests from one box of OvuQuick One-Step Ovulation be used during the same month with tests from another box of OvuQuick One-Step Ovulation?
Yes
9. Can I store my urine and test later?
*Your urine can be stored at room temperature 59–86°F (15–30°C) for up to 8 hours, or in the refrigerator 36–46°F (2–8°C) for up to 24 hours. **Do not freeze it.** If you stored your urine, sometimes there will be cloudy urine at the bottom of the cup. Use the clear urine from the top. Warm your urine to room temperature after refrigeration.*
10. Will Clomid[®] or Serophene[®] affect my test results?
If you are taking Clomid[®] or Serophene[®], follow your doctor's instructions as to when to begin testing. Testing too soon may cause a false positive (false surge) result.
11. I've used all the tests as instructed, but I have not yet detected an LH surge. What do I do?
Six (6) tests are sufficient for many women with regular cycles (if your cycle varies by fewer than 3 days). If your cycle varies by 4 days or more, you may need to test with the 9-day kit. If you still do not detect your LH surge, you may want to test again during the next menstrual cycle or contact your physician or reproductive specialist.

NOTES

Listed below are possible reasons why you did not detect an LH surge:

1. You may not have had an LH surge during this cycle, or did not ovulate mid cycle.
2. You may have tested too early or too late in your cycle.
3. The amount of LH in your urine may have been too low to detect.

ASSISTANCE

If you have additional questions, call our Technical Support Line at +1 866 848 7687 or 858-824-0888 (local), between 7:00 am and 5:00 pm, Monday through Friday, info@vitrolife.com

REFERENCE

1. Elkind-Hirsch, K; Goldzieher, JW; Gibbons, WE and Besch, PK. *Obstetrics and Gynecology*, 67(3): 450-453, 1986.

REF

15100 – OvuQuick One-Step Ovulation Predictor – 6 test
15101 – OvuQuick One-Step Ovulation Predictor – 9 test
OvuQuick[®] is a registered trademark of Vitrolife
Clomid[®] is a registered trademark of Merrel Dow Pharmaceuticals, Inc.
Serophene[®] is a registered trademark of Sero Laboratories, Inc.

*After collecting urine

Manufactured for:
Vitrolife Inc.
4940 Carroll Canyon Rd., Suite 100
San Diego, California 92121 USA
Tel: +1 866 848 7687
www.vitrolife.com

Vitrolife

Index of Symbols

	Attention, see instructions for use
	For <i>in vitro</i> diagnostic use only
	Store between 2–30°C (35–86°F)
	Tests per kit
	Use by
	Lot Number
	Do not reuse
	Catalog #
	Consult Instructions Before Use

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Clomid[®] es una marca registrada de Merrel Dow Pharmaceuticals, Inc.
Serophene[®] es una marca registrada de Serono Laboratories, Inc.